


1. Start a new case

STEMI: ED Patient

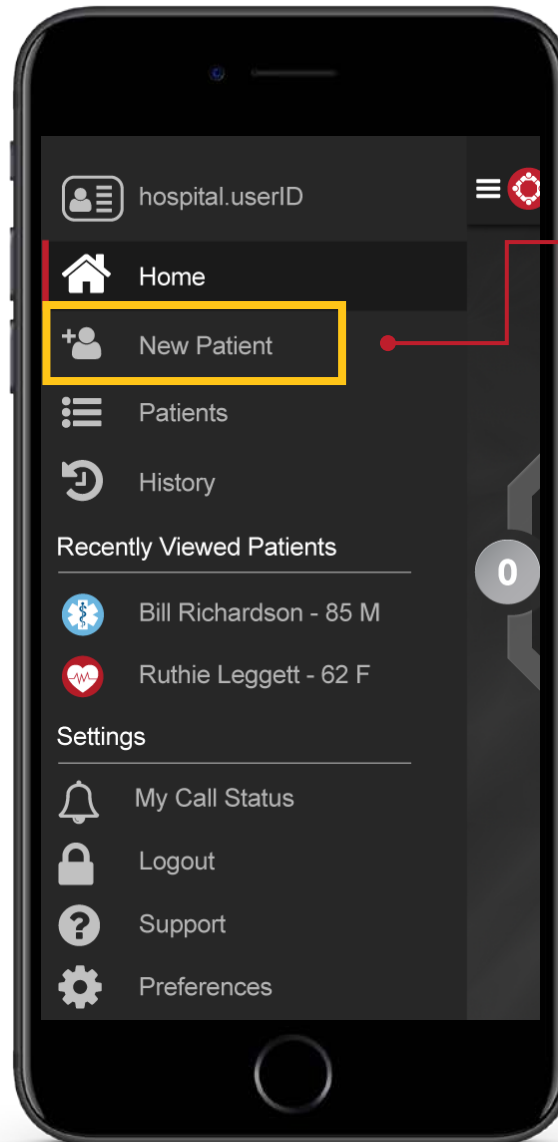

New Patient


From the the Navigation Panel, tap +New Patient to start a new case.

2. Select your module:

STEMI: ED Patient


Select a Patient Type

Tap STEMI to start a new STEMI Case.

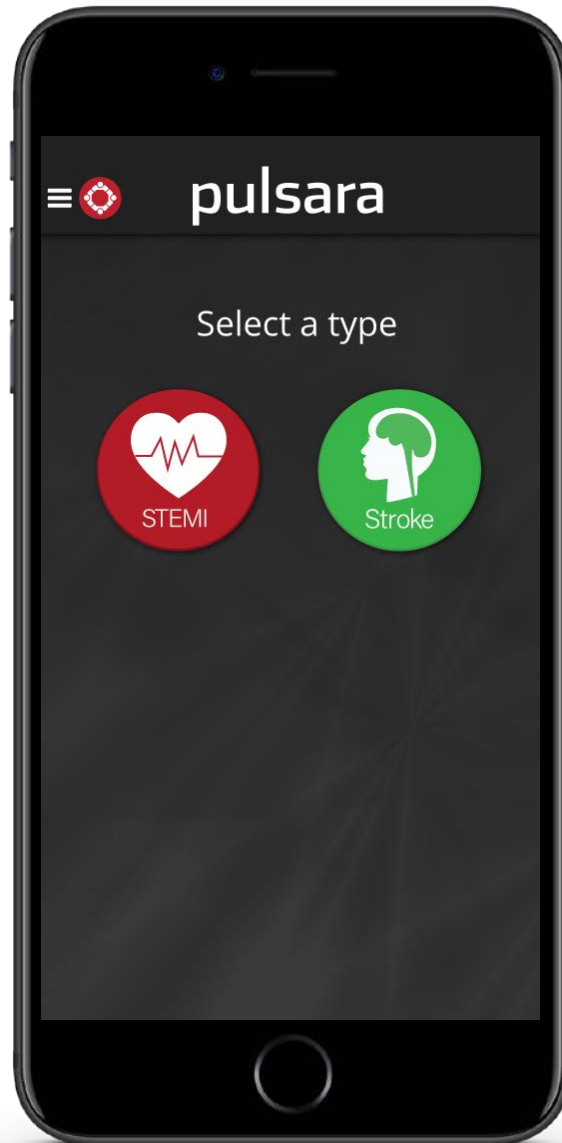

3. Enter your case details:

STEMI: ED Patient

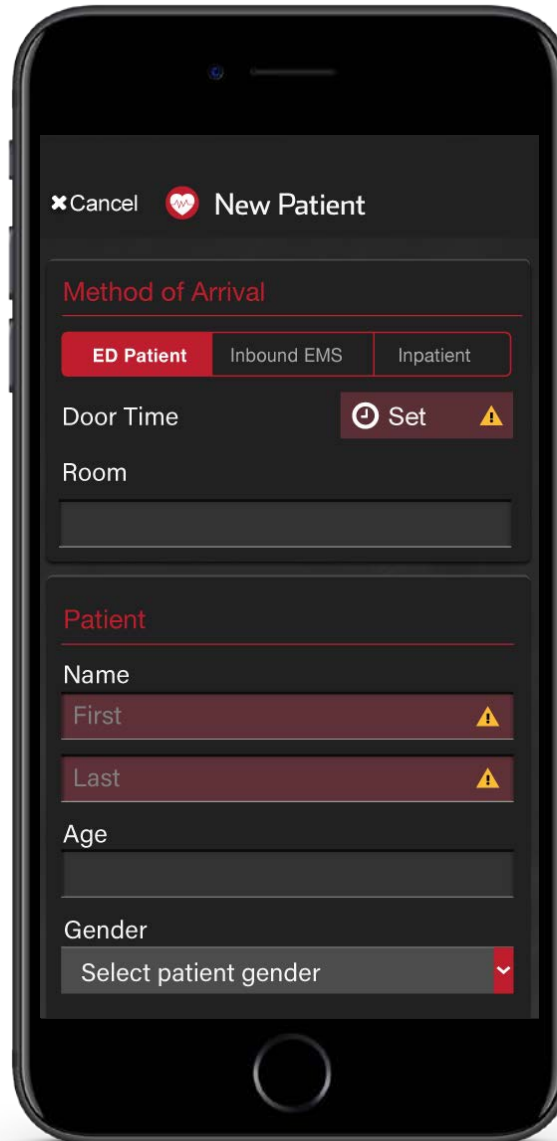

The smartphone screen displays the 'New Patient' form. At the top, there is a 'Cancel' button and a heart icon. Below this, the title 'New Patient' is shown. The form is divided into two main sections: 'Method of Arrival' and 'Patient'. The 'Method of Arrival' section has three buttons: 'ED Patient' (highlighted in red), 'Inbound EMS', and 'Inpatient'. Below these are fields for 'Door Time' (with a clock icon and a 'Set' button), 'Room' (with a warning icon), and a text input field. The 'Patient' section has fields for 'Name' (with 'First' and 'Last' sub-fields, both highlighted in red and with warning icons), 'Age', and 'Gender' (with a dropdown menu labeled 'Select patient gender').


Required Patient Information

All fields highlighted in red are required.

- 1 Door Time
- 2 Room Number
- 3 Patients First and Last Name
- 4 Age
- 5 Gender

4. Activate the case:

STEMI: ED Patient

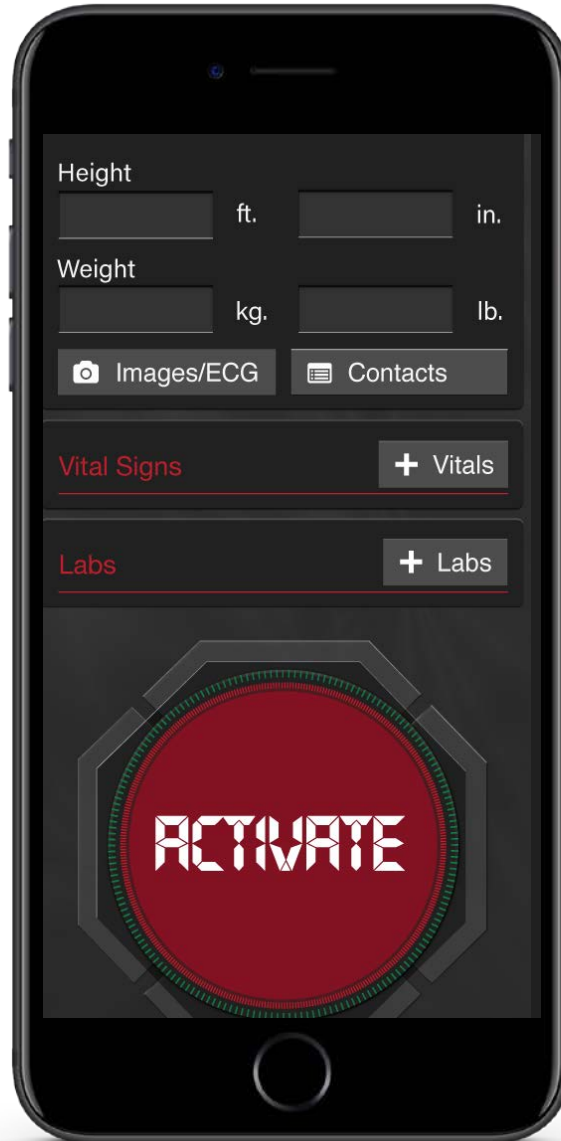

Optional Patient Information

NOTE: most patient demographic fields are optional. Enter only information that is critical to share with the entire team.

- 1 Height
- 2 Weight
- 3 Picture of ECG, Drivers License, ETC.
- 4 Vitals
- 5 Labs
- 6 **TAP ACTIVATE to start the case**